



# Bio-Synthesis® quick start guide

1. Read operation manual prior to using the equipment.
2. Do not use the unit if any contraindications are present.
3. Place on a sturdy, even surface appropriate for weight of unit.



4. Plug cord from arm into the corresponding receptacle in mounting post.

5. Mount arm into the mounting post and hand tighten **SCREW** at base of post.



6. Plug **POWER CORD** into base and into standard wall outlet appropriate for the unit.

7. Click **SWITCH** at rear of unit to ON position.



To wall →



8. Touch screen on front of unit to access menu.



9. Apply eye protection for client (eye pads and goggles) and unit operator (goggles).

10. After service, click switch at rear of unit to OFF position.