



Bio-Therapeutic
Skin Care for life inside and out™

bt-accent® AIR quick start guide

1. Read operation manual prior to using the equipment.
2. Do not use the unit if any contraindications are present.



3. Attach accessory holster to back of unit. Secure with four screws and allen wrench provided.

4. Attach Handpiece tube into JET connection on top of unit and place Handpiece in holster.



5. Attach AIRbrush tube into WAND connection on top of unit and place in holster.



6. Attach Dome Tube onto Dome Tube Connector located on front of machine above Flow Meter.



7. Attach the other end of Dome Tube to Dome.

8. Attach **POWER CORD** to unit, plug into standard wall outlet appropriate for the unit.

Press **POWER SWITCH** at rear of unit to "On".



To wall →



9. Press Power button at top of unit near handle.



10. **DOME**

Select **DOME** to activate AIR.

11. Adjust flow meter to desired level.



12. AIRbrush

Fill cup half full with RESTYFLUID, or other appropriate bt-ceuticals® product.



13. Select **WAND**

Press trigger to start spray.

When empty, allow AIR to flow through AIRbrush for one minute. Follow cleaning instructions found in the Operation Manual.



14. Handpiece

Select Handpiece Tip.



15. Select **JET**.



16. Turn off by pressing selected accessory button again.
Press power switch at the rear of the unit to "Off."