



Bio-Therapeutic
Skin Care for life inside and out™

bt-accent® LED quick start guide

1. Read operation manual prior to using the equipment.
2. Do not use the unit if any contraindications are present.



3. Mount the arm into the mounting post on top of unit.

4. Hand tighten **SCREW** to stabilize arm.



4. Plug the **CABLE** from arm into receptacle on back of unit base. Align groove in the plug with groove in receptacle and hand tighten.



5. Plug **POWER CORD** into base and into standard wall outlet appropriate for the unit.

To wall →

6. Press **POWER** button.



7. Select **COLOR**: Red, Blue, or Combination.
8. Select **RATE**: 10, 40 or 100.
9. Select **TIME**: 15, 20, or 30 minutes.

10. Apply eye protection for client (eye pads and goggles) and unit operator (goggles).



11. Press **START**.