



Bio-Therapeutic
Skin Care for life inside and out™

bt-analyze® quick start guide

1. Read operation manual prior to using the equipment.
2. Do not use the unit if any contraindications are present.



3. Turn on the bt-analyze by holding the power button (A) for 2-3 seconds. Proceed by pressing the memory button (B). The unit will calibrate itself.

4. Place the bt-analyze on the surface of the skin. A beep will signify the reading is complete. During the reading, do NOT press the power or memory buttons.



5. Press the memory button to save the reading at the top of the display screen. Readings will save in the order you perform them.





6. Follow steps 4 and 5 on the 7 zones of the face.

By the numbers:

Range: 0 - 65

Dry: 25 or below

Optimum: 35 - 45

High level of hydration: 55+

Reminder:

A soiled sensor slows calibration and reading times. Wipe the sensor without pressing down with a clean, dry tissue between each reading and with an alcohol wipe after each use. Always remember to replace the cover when not in use.

The bt-analyze sensor will make a slight click when depressed during moisture readings; this is normal. A reading of “UUU” means an error has occurred.

Wipe down your bt-analyze with Opticide-3 wipes after each use.