



Bio-Therapeutic
Skin Care for life inside and out™

bt-nano® quick start guide

1. Read operation manual prior to use
2. Do not use the unit if any contraindications are present



3. Press **ON/RESET** button.
Wait for welcome screen to disappear.

4. Press **MODE** button to access facial choices. Watch the top line of text.
5. Press **PROGRAM** button to select facial preferences. Watch the second line of text.



6. Press **OPTION** button to select what number of service the client is on.

In some cases, options will be High or Low instead of a number. Choose Low for sensitive skin and clients beginning a series.



7. Press **RUN/PAUSE** to begin your service

8. Unit will shut off on its own after two minutes of non use or after the service finishes.