



# bt-vision® quick start guide



1. Read manual prior to using the equipment.
2. Insert 3 AAA batteries in battery compartment under visor.

3. Choose and install preferred lenses by gently pushing each lens into insertion clasps under visor.
  - a. Lenses should be inserted with flat side towards eyes.
  - b. Installing two lenses allow increased magnification when using both.



4. Place visor on head with band running horizontally across forehead just above brows.





5. Hold visor in place and tighten band by pressing the gray tab and pulling the tail until fit is snug but not tight.

6. To adjust visor, loosen screws on both sides of bt-vision®, move visor, then tighten screws.



7. Turn the light on by moving the light switch forward. The **LIGHT SWITCH** is on the top left side of visor.
8. Adjust light intensity by moving the **SMALL SWITCH** directly on light.