



**Bio-Therapeutic**  
Skin Care for life inside and out™

# Bio-Brasion® TRINITY quick start guide

1. Read operation manual prior to using the equipment.
2. Do not use the unit if any contraindications are present.



To wall →

3. Attach power cord to unit and then to standard wall outlet.

4. Attach Waste Container to back of unit. Make sure **RED WASHER** is in place and hand tighten.



5. Attach Safety Filter Cup to back of unit. Make sure **BLACK WASHER** is in place and hand tighten.



6. Prepare Handpiece: Make sure **RED WASHER** is in place, and that **SILICONE SLEEVE** is over Handpiece.

7. Attach tubing:

Insert tube into corresponding openings on side of unit until firmly attached.

Push other end of tube onto corresponding extension of Handpiece and hand tighten fittings.



8. Place Diamond Tips in Diamond Tip holder.

9. Choose appropriate Diamond Tip according to skin type and insert into Handpiece.



10. Put open Handpiece Sheath over Handpiece.



11. Flip **SWITCH** at back of unit to "On."



12. Press Power.

13. Select Low, Medium, or High Vacuum level.



14. Lower Diamond Tip numbers and higher Vacuum setting will increase the level of over all exfoliation, as will speed of movement. Please consider all factors when choosing Diamond Tips and settings.